**MANUALE D'USO / USER'S GUIDE****INDICE / TABLE OF CONTENTS**

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*Shaker vibrators guarantee a notable power changeable by means of an electronic circuit and anti-trouble filter. They are assembled in accordance with safety rules (protection fuse, schuko plug, green bright switch). They have a removable rubber surface with counter-sink edges for an easy cleaning even with steam (high thermic and mechanical resistance of the materials).*

*Shaker vibrators guarantee an high noiseless and steadiness by means of a perfect vertical movement, deadened by four little feet made with soft mousse. Solid iron structure (2mm.) with removable front little mask made of anticorrosive metal, that allows an easy maintenance if necessary.*

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The vibrator is shipped in an adequate package that requires the common handling cautions for fragile materials, as shown on the package itself. Use only the original package if the equipment must be returned to the manufacturer.  
To carry the vibrator without package, keep it in horizontal position and use the base as holding point.

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Remove the equipment from the package handling with care. Check the power supply system is correctly grounded.  
Place the vibrator on a level surface so that air can freely circulate under the equipment bottom side.  
Insert the safety plug of the power supply cable, firstly into the electric socket and then into the electric system, after being sure the voltage corresponds to the one selected on the device.

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- Turn the knob (1-fig.1) counter-clockwise to the minimum position.
- Press the main green switch (2-fig.1) on the "T".  
Check the green light of the switch is on.
- Turn the knob (1-fig.1) to the required power.
- At the end of your work, press the switch on "Q".

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Do not modify, change or put out of order any electrical or thermal safety protection of the equipment.

The user must not attempt to carry out any repair unless authorized.

Do not clean or repair the vibrator while it is connected to an electrical system.

Regularly check if the power cable is damaged.

When authorized technicians decide to replace spare parts, they must use original replacements and moreover they must adhere to the manufacturer's information and technical characteristics.

In case of any doubt about safety rules, please contact the manufacturer or local distributor before using the equipment.

However use the vibrator in accordance with the general safety and accident prevention rules of the country in which the equipment is installed.

#### ATTENTION:

avoid to use the vibrator in an environment where the explosive gases may be present.

Place the vibrator on a level surface so that air can freely circulate under the equipment bottom side. Do not clean, move or repair the vibrator while it is connected to an electrical system.

#### 7) Manufacturer's responsibility.

The vibrator must be used only for its designed purpose.

Improper use of the equipment is to the user's own risk and sole responsibility.

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*The warranty covers only equipment with production and natural defects as indicated in the certificate that comes with the device.*

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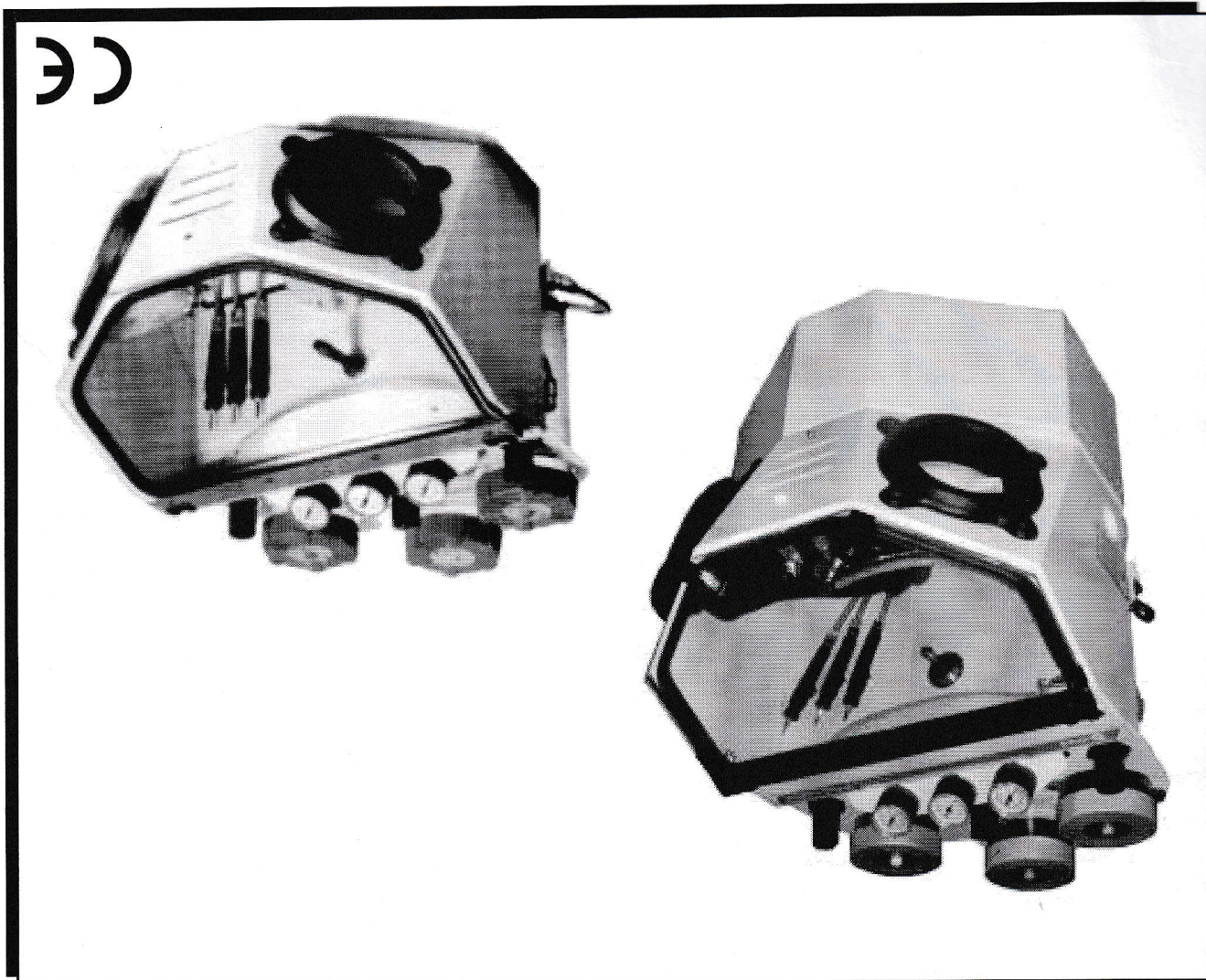
**ATTENTION:**  
*the numbers on the graph (3-fig.1) for the adjustment knob, are indicative only.*

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